

avoset™

Medication reservoir & extension administration set

For use with Avoset™ Infusion Pump

Step 1 - Filling the medication reservoir


1

Visually inspect the packaging for breaches of integrity prior to use.



2

Ensure clamp is open and white release clip in place. Do not remove white release clip until attachment of set to pump.

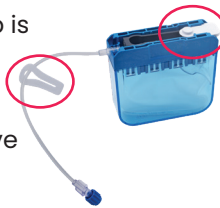

3

Draw required fluid using a syringe or other filling device and diluent.



4

Remove protective vented cap from the tubing luer connector.



5

Connect filled syringe or filling device to the medication reservoir luer connector.



6

Fill medication bag with the medication in the syringe or filling device.*



7

Remove excess air: Tilt medication reservoir and withdraw air through exit port of the bag.



8

Close clamp.



9

Disconnect syringe or filling device from luer. Cap luer with red, non-vented stopper provided, or attach extension set.



*Recommendation: To avoid overfilling with fluids and air, fill the medication reservoir until half full, then remove the excess air and continue to fill the reservoir with remaining fluid.



avoset™
Quick Guide



Step 2 - Attaching an extension set to the medication reservoir


1

Open the extension package and visually inspect the packaging for breaches of integrity prior to use.



2

Remove extension set upstream transparent luer cap.

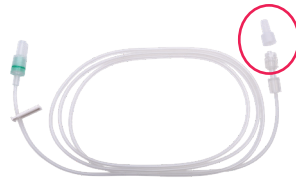


3

Remove the red luer cap from the medication reservoir tube.



4

Twist the extension set transparent luer into the medication reservoir luer and connect them.

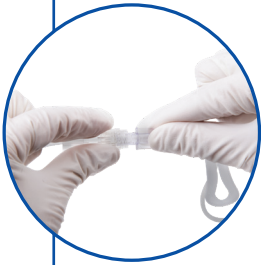

5

The complete set is ready to be primed.



Step 3 - Attaching the medication reservoir to the pump

1

If present, remove the white anti-free flow release clip from the new administration set cassette.



2

Position the medication reservoir at the bottom of the pump, groove front facing (aligned with pump's shape)



3

Push the cassette into the pump's cassette socket. You should hear 2 clicks to indicate that the cassette is fully attached. Verify that it is securely attached.



Step 4 - Priming the medication reservoir with the pump

- 1 Make sure that fresh batteries have been installed in the pump, pump is ON, and an administration set is attached to the pump.
- 2 Make sure the administration set is disconnected from the patient.
- 3 Press PRIME hard key and follow the pump onscreen instructions.
- 4 Close all administration set clamps.

